
ThevoautoActiv

Ⓛ	Bedienerhandbuch – ThevoautoActiv	4
Ⓛ	User manual – ThevoautoActiv	8
Ⓛ	Gebruiksaanwijzing – ThevoautoActiv	12
Ⓛ	Brugermanual – ThevoautoActiv	16
Ⓛ	Instrukcja obsługi – ThevoautoActiv	16

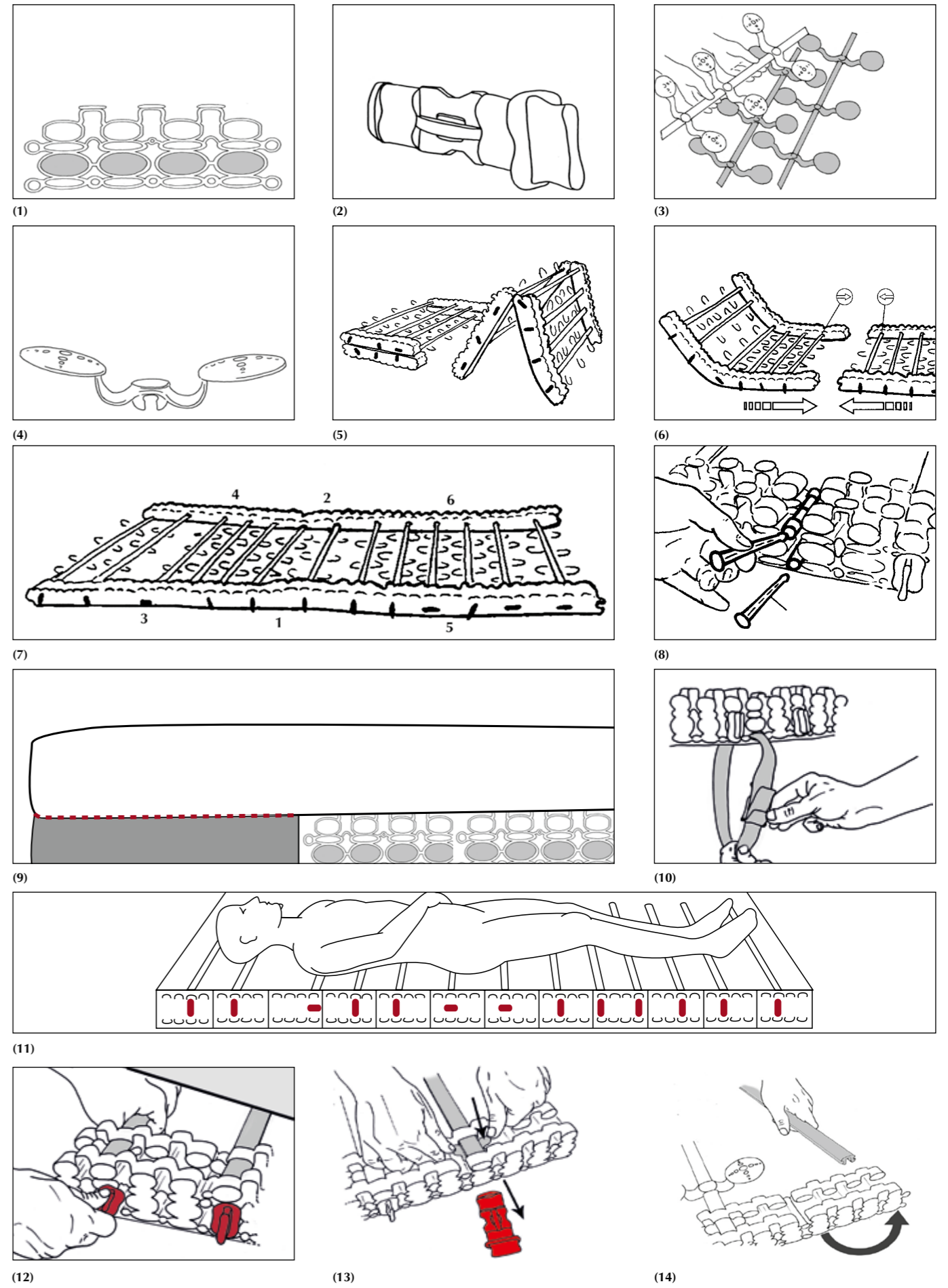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Dear client,

with ThevoautoActiv you have purchased a mobility bed, which meets the highest quality demands of home and hospital care. For ThevoautoActiv we have used high-quality materials, which guarantee a long durability.

Before using ThevoautoActiv you should absolutely read this user manual carefully. There are a lot of information and details. To obtain the best possible therapeutic benefit, the system has to be adjusted to every individual patient.

If you have any questions or comments on ThevoautoActiv, please do not hesitate to contact us:
+49 (0) 4761/ 886-68 or -63

Further information on the prevention and treatment of pressure sores you will find on our website at **www.thevo.info**.

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Content

	Page
Prevention and treatment of pressure sore.....	9
Components	9
Assembly	9
Standard setting at delivery	10
Methods of disinfection and cleaning advises.....	10
Application risks and contra indications.....	10
Conditions of guarantee.....	10
Technical data of the complete system	10
Advanced improvements	11
Adjusting the adaptors	10
Adjusting the FGF- track	11

Prevention and treatment of pressure sore

ThevoautoActiv has been developed for the prevention and treatment of pressure sores **up to and including stadium III according to EPUAP** (European Pressure Ulcer Advisory Panel). **ThevoautoActiv is characterised by its individual adaptation to the respective patient and his clinical picture.**

An individual adaptation is facilitated by:

- 24 activ side elements (ASE)
- 26 adaptors
- 13 flexible fibreglass-tracks
- 46 wing suspensions at a width of 90 cm (35.4") or
- 52 wing suspensions at a width of 100 cm (39.4")

ThevoautoActiv is mainly suited for the care of patients with neurological clinical pictures and pain symptoms. Besides an excellent pressure distribution, the system is well suited for the perception and movement support.

Components

Active side element (img. 1)

The active side element does not only connect the single flexible fibreglass-track, but also **offers the possibility to integrate ThevoautoActiv into common nursing and hospital beds.** You are able to carry on using the head and foot adjustment. The system is simply laid on the existing slatted frame or inlay frame. Due to the flexible construction you can still use the head and foot adjustment as well as the height adjustment of the bed.

Adaptor (img. 2)

The adaptors are situated in the active side element (ASE). Through these adaptors the spring travel of the flexible fibreglass-track can be restricted as required. Thus, through various adjustments body parts at risk can be alleviated additionally. By taking the adaptors completely off, the highest possible alleviation is reached.

Flexible fibreglass-track (img. 3)

These tracks are made from a durable material by innovative glass fibre technology (pultrusion). Beneath a long durability they ensure an optimum lying comfort, which has a favourable effect on the convalescence of the patient.

Wing suspension (img. 4)

3 or 4 wing suspensions situated on one flexible fibreglass-track optimally adapt to the body contours of the patient. An anatomically correct body alignment is ensured and sleep is influenced positively. Each wing suspension can be moved or removed in order to support the individual preventive and therapeutic treatment in an optimum way.

Assembly

Please note, that the ThevoautoActiv mattress will be delivered in an extra carton. The system will be delivered in two parts (img. 5).

1. Please put the two parts close together, so that the side elements fit into each other. You should always pay attention on the direction marking (img. 6).
2. The connection thorns have to be placed tight and flush into the side elements at the connection points 1 to 6 (img. 7 and 8).
3. Lay the under frame on the existing slatted frame or inlay frame of your bed. Please pay attention to the label for the head and foot section.
4. Now lay the mattress on the under frame.

Important hint: ThevoautoActiv can also be used without a pre-existing slatted frame. In this case the supporting surfaces along the sides of the bed base must be at least 10 cm (3.9 ") wide.

Important hint: If you have a bed with side rails you will probably need to use side rail extensions.

5. The black fixation flaps of the mattress cover in the head and foot section must be flapped around the under frame and closed with the zipper (img. 9).
6. The under frame of ThevoautoActiv is safely fixed to the bed frame with the 6 belts. Thread the belts through the bottom holes of the active side element and then around the tubular frame of the bed. Thread the belt through the buckle and pull it tight. To release the belt, open the buckle and pull out the belt (img. 10).

Attention: If you have a height adjustable bed, please fix the belts to the lifting system.

7. The system can now be individually adjusted to meet the needs of the respective patient.
8. Check the settings after 24 hours at the latest and carry out eventually necessary changes.

Standard setting at delivery

At the delivery of ThevoAutoactiv the adaptors and flexible fibreglass-tracks are set as shown on img. 11. Additional adjustment see on page 11

Important hint: The under frame should be used in combination with the corresponding mattresses only. Please pay attention to the following:

- ThevoautoActiv 50 up to 50 kg (110.2 lbs) patient weight
- ThevoautoActiv 100 from 50 to 100 kg (110.2 to 220.5 lbs) patient weight
- ThevoautoActiv 135 from 100 to 135 kg (220.5 to 297.6 lbs) patient weight

In case of a heavily distinctive unequal distribution of the body weight (i.e. strong pelvis or stomach, leg amputation) a mattress for higher weight should be used.

Methods of disinfection and cleaning advises

Matress

Due to the zip the covers can be taken off very easily. They can be washed at 65 °C (149 °F) and are suitable for tumble dryers. All mattresses are autoclavable and therefore suited for re-use.

Jersey cover

- washable up to 65 °C (149 °F) • suitable for tumble dryers • breathable

Incontinence cover

- washable up to 65 °C (149 °F) • suitable for tumble dryers • to be cleaned by wiping disinfection

Under frame

Simple cleaning: the system can be cleaned with non-corrosive detergent. Please ensure that the system will be dried carefully after every cleaning.

The system can also be cleaned with disinfectant on alcohol basis.

Disinfection

For the disinfection we recommend two methods accepted by the Robert-Koch-Institute:

- the fractionated vacuum method (VDV-method) up to a max. temperature of 110 °C (230 °F).
- the steam flow method up to a temperature of 110 °C (230 °F).

Important hint: A thermal treatment over 110 °C (230 °F) causes a premature ageing and deformation of the plastic parts and should therefore be avoided. Please make sure when using chemical disinfectants that they are suitable for plastic.

Application risks and contra indications

Referring to the previous practical use and several tests executed, we are not knowledgeable of any problem in the use of this product. When used properly, risks can be excluded. No contra indications are known.

Conditions of guarantee

Thomashilfen takes full responsibility for the quality of ThevoautoActiv. All materials as well as the processing have been chosen under the highest production standards. All systems have been tested very carefully before leaving the factory. Thomashilfen offers a guarantee of 24 months. Therefore you should keep all documents on ThevoautoActiv very carefully. The guarantee covers all claims on the basis of production or material failures. Excepted are wear or tear caused by over or inappropriate use.

Important hint: ThevoautoActiv is suited for re-use. After each patient change the complete system should be cleaned and disinfected according to one of the cleaning methods stated above. A maintenance by the producer is not necessary.

Technical Data of the complete system

Outer dimensions ThevoautoActiv	90 x 200 cm/ 35.4 x 78.7 "	Height	100 x 200 cm/ 39.4 x 78.7 "	Height
up to 50 kg (110.2 lb)	21.4 kg/ 47.2 lbs	19 cm/ 7.5 "	23,2 kg / 51.2 lbs	19 cm/ 7.5 "
50 to 100 kg (110.2 to 220.5 lb)	22.4 kg/ 49.4 lbs		24,3 kg / 53.6 lbs	
100 to 135 kg (220.5 to 297.6 lb)	24.2 kg/ 53.4 lbs		26,3 kg / 58 lbs	

Other measurements on request.

Advanced improvements

On the following pages we will show you nine setting options of which you shall choose the right one for your patient in order to accordingly set ThevoautoActiv. These recommendations are based on the results of our carefully checked experiences and for that reason they shall be viewed as average values. Please keep in mind that we can only give recommendations for the individual. The individual adjustment should be made because of the clinical picture. We recommend a regular check of the settings to ensure also the correct treatment in the progressing therapy. When adjusting the adaptors please act step-by-step from the head to the foot and compare your personal setting with the corresponding picture in this manual. The adjustment of the adaptors should be carried out as described on.

Important hint: Only remove the adaptors and not the flexible fibreglass-tracks.

Setting recommendation for a body height of 185 - 200 cm (72.8 – 78.8")

- Light weight, light physique (img. 13)
- Medium weight, normal physique (img. 14)
- Heavy weight, strong physique (img. 15)

Setting recommendation for a body height of 170 - 184 cm (66.9 – 72.4")

- Light weight, light physique (img. 16)
- Medium weight, normal physique (img. 17)
- Heavy weight, strong physique (img. 18)

Setting recommendation for a body height of 150 - 169 cm (59.1 – 66.5")

- Light weight, light physique (img. 19)
- Medium weight, normal physique (img. 20)
- Heavy weight, strong physique (img. 21)

Important hint: If the person is lighter than 30 kg (68 lb) please get in contact with us.

Adjusting the adaptors

In order to adjust the adaptors you have to grab under the mattress and pull the flexible fibreglass-track slightly upwards. Now you can turn the adaptor to the required direction or rather remove it completely. Please ensure that the adaptor grips tight into the stabiliser after every adjustment. Only in this case an unintended turning can be avoided (img. 12).

Adjusting the FGF- track

First you will have to take the adaptor out off the side element. Push the FGF- track out off the stabiliser (the stabiliser is situated under the FGF- track in the side element) (img. 13). Now you can take the FGF- track out off the side element. Put the stabiliser in the desired position of the side element and slide in the FGF- track (img. 14). Put in the adaptor again and turn it into the required position.

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