

ThevoSchlummerstern A

| | | |
|----|---|----|
| DE | Bedienerhandbuch – ThevoSchlummerstern A..... | 5 |
| GB | User manual – ThevoSleepingStar A..... | 12 |
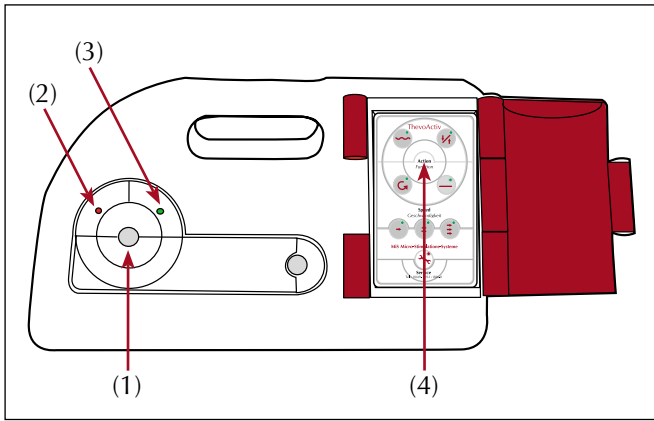


(1)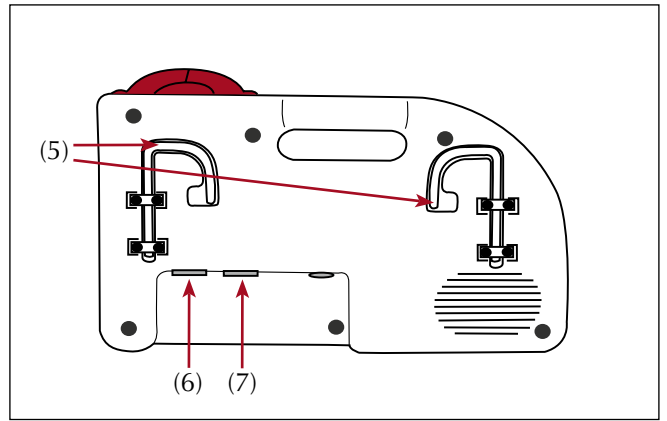


(2)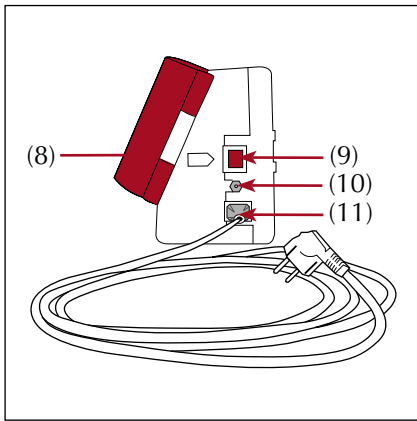


(3)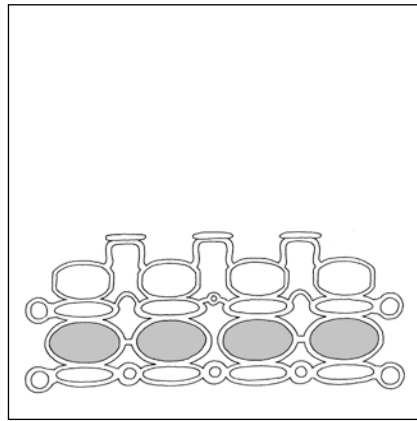


(4)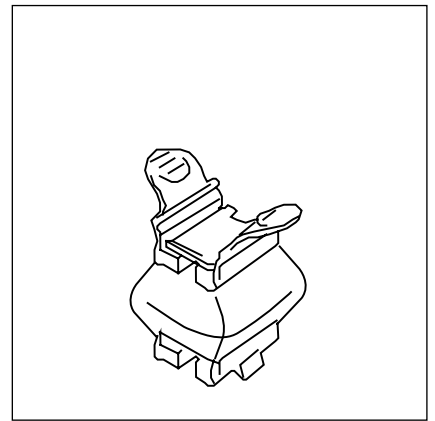


(5)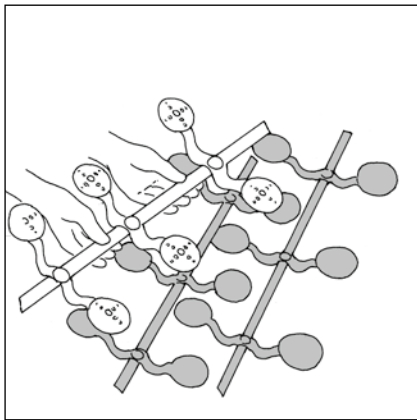


(6)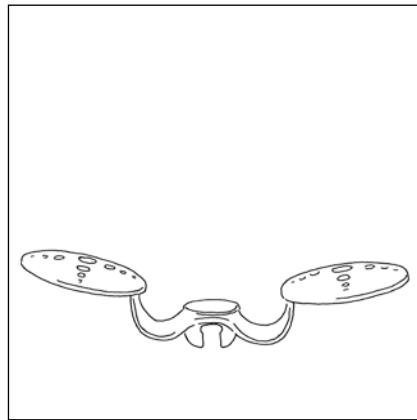


(7)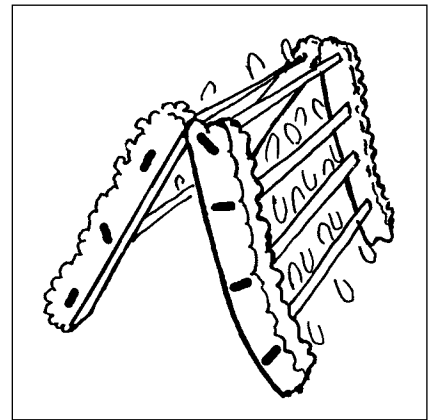


(8)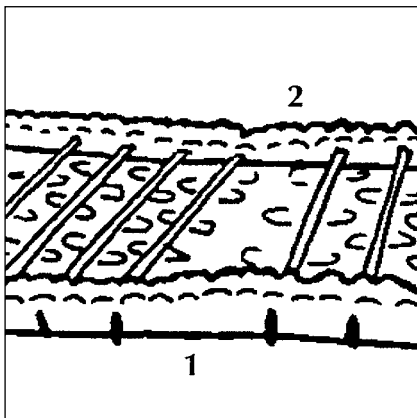


(9)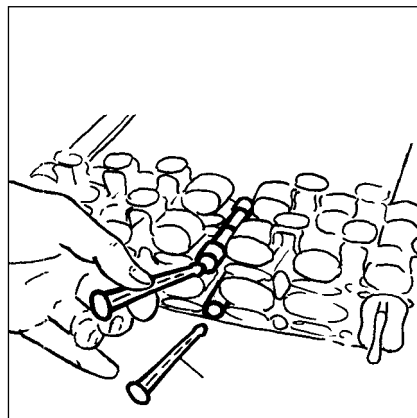


(10)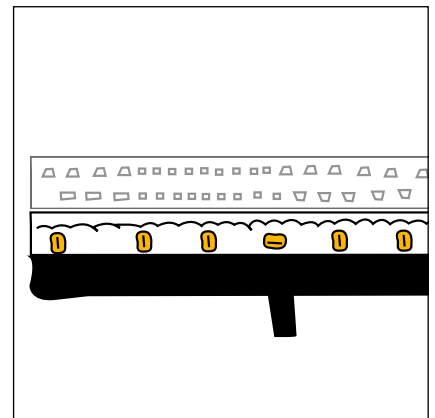


(11)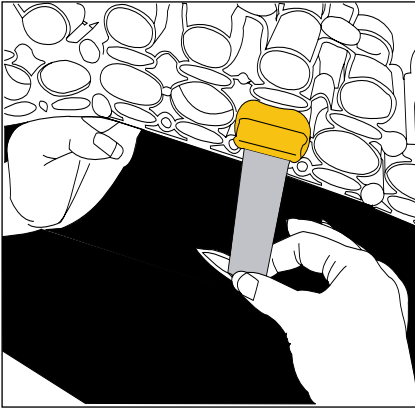


(12)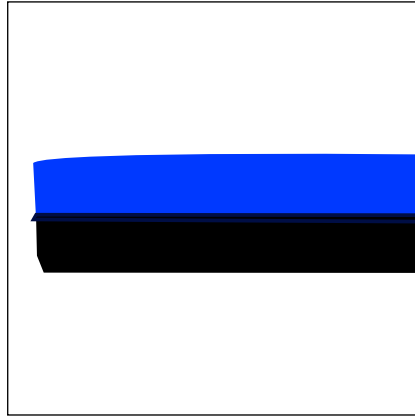


(13)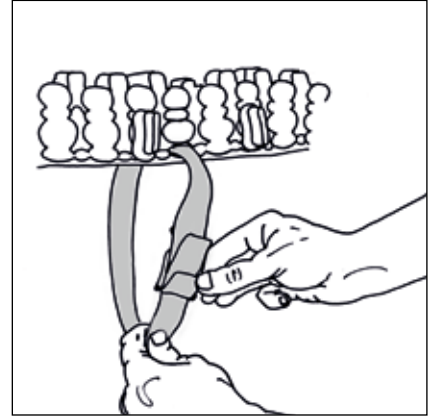


(14)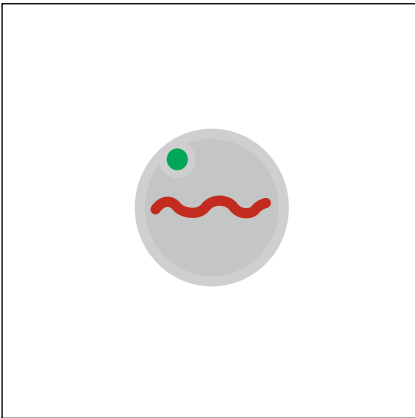


(15)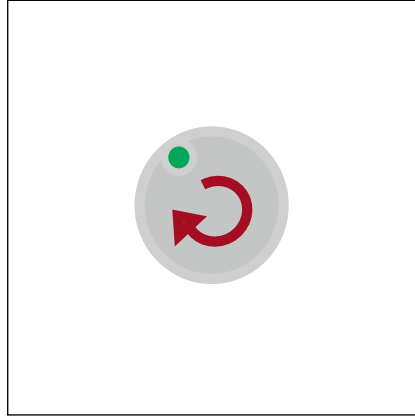

(16)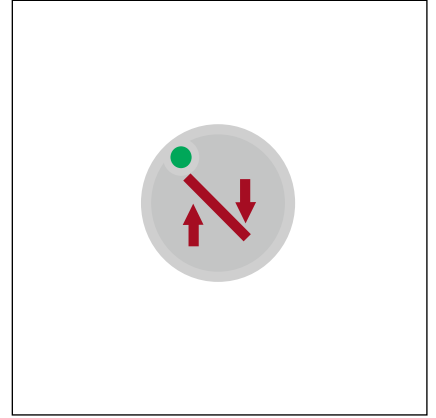

(17)

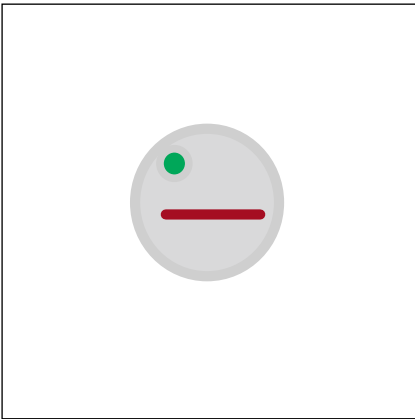(18)

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Dear client,

with ThevoSleepingStar A you have purchased a therapy system for ill and disabled children, which meets the highest quality demands of home and hospital care. For ThevoSleepingStar A we have used high-quality materials, to ensure long durability.

Before using ThevoSleepingStar A, we recommend that you read this user manual carefully. It contains a lot of helpful information and hints.

If you have any questions or comments, please do not hesitate to contact us.

Please visit our website for further information on the prevention and treatment of pressure sores at **www.thevo.info**

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Contents

Page

| | |
|---|----|
| Sleep disorders of disabled children | 13 |
| Clinical pictures / Contraindications..... | 13 |
| Components | 13 |
| Before starting up / assembling the system | 14 |
| Start-up of ThevoSleepingStar A..... | 14 |
| Changing the movement pattern | 14 |
| The movement patterns of ThevoSleepingStar A | 14 |
| Additional valves | 15 |
| Disinfection methods and cleaning recommendations | 15 |
| Maintenance | 15 |
| Self test / service LED | 16 |
| Conditions of warranty | 16 |
| Safety hints | 16 |
| Disposal of the system..... | 17 |
| Technical data | 17 |

Sleep disorders of disabled children

ThevoSleepingStar A is a positioning and therapeutic system suitable for children, which has been developed to ease the nightly care of ill and disabled children.

It can be used to support body perception of disabled children, to support good pain therapy, to develop a physiological sleep pattern, for the inhibition of muscle tensions and, for the prevention and treatment of pressure ulcers **up to and including grade IV according to EPUAP** (European Pressure Ulcer Advisory Panel).

Fields of application:

- Perception-, movement-, and development support
- Treatment of sleep disorders of disabled children
- Prevention of contractures
- Inhibition of spasticity
- Support of pain therapy
- Preservation of body image
- Supports / promotes a physiological sleep architecture
- Prevention and treatment of pressure ulcers (**up to and incl. grade IV acc. to EPUAP**)
- Pain patients, e.g.
 - Rheumatism
 - Soft tissue rheumatism
 - Tumour patients

Clinical pictures:

Pain, coma vigil, coma, craniocerebral injury, spina bifida, cerebral palsy and similar clinical pictures, oncological diseases, genetic skin diseases.

Contraindications

The use of ThevoSleepingStar A is not recommended for the following clinical pictures:

- Lyell's Syndrome
- Status after rotation flap plastics
- Large area 3rd degree burns at the back

Information: Referring to the previous practical use and several tests executed, we are not knowledgeable of any problem in the use of this product.

Components

Control unit

ThevoSleepingStar A is operated by a control unit. The integrated microprocessor controls the movement of the actors. The control unit can be hung up to the bed with two retainers (img. 2, 5).

Via the keypad (img. 1, 4) different movement patterns can be set according to the demands of the child. This ensures individual prevention and treatment of pressure sores and support of perception.

Control unit, front (img. 1)

Start / Stop-switch (1), Failure indication (2), Status of operation (3), keypad (4)

Control unit, back (img. 2)

Fixation clip (5), Port left bed side (6), Port right bed side (7)

Control unit, sideways (img. 3)

Key pad (8), Power switch (9), Pump port (10), Power plug (11)

Active side element (img. 4)

The active side element does not only connect the single flexible fibreglass-tracks. It also offers the possibility to integrate ThevoSleepingStar A **into every hospital, children, or juvenile bed**. Unlike rigid under frames the head and foot adjustments of the bed can still be carried out.

Valves (img. 5)

The valves are small bellows integrated in the active side elements. Individually adjustable movement patterns are achieved by feeding air through the supply pipe which slightly lifts the flexible fibreglass-tracks. This results in a relief of pressure in the vulnerable areas.

Flexible fibreglass-track (img. 6)

These tracks are made from durable material by innovative glass fibre technology (pultrusion). In addition to a long durability, they ensure optimal lying comfort, which has a favourable effect on the recreation of the child.

Wing suspension (img. 7)

3 or 4 wing suspensions situated on one flexible fibreglass-track optimally adapt to the body contours of the child. An anatomically correct body alignment is ensured and sleep is influenced positively. Each wing suspension can be moved or removed in order to support the individual preventive and therapeutic treatment in an optimum way.

Before starting up / assembling the system

Before starting up ThevoSleepingStar A, please follow below stated worksteps:

1. Plug the 2 cable harnesses of the system into the sockets on the back of the control unit. Right bed side (img. 2, 6), left bed side (img. 2, 7).
2. Plug the air hose into the pump port (img. 3, 10) sideways at the control unit.
3. Plug the power plug in (img. 3, 11).

ThevoSleepingStar A is now ready for operation and for being used for the prevention and treatment of pressure ulcers. The start-up of the different functions will be described in the following chapters. Please read them carefully to ensure optimum prevention or treatment and to avoid failures.

Start-up of ThevoSleepingStar A

a) Assembly

1. Please note, that the ThevoSleepingStar A mattress will be delivered in an extra box. The underframe is folded in the middle. Lay the underframe flat on the floor, ensure that the side elements interlock (img. 9)
2. The connection thorns have to be placed tight and flush into the side elements at the connection points 1 and 2. (img. 10)
3. Lay the black part of the cover on the existing slatted frame or inlay frame of the bed. Put the underframe on the cover. (img. 11) Please pay attention to the label for the head and foot section. Pay attention that the opening of the zipper is on the foot section.
4. Pull the belt straps through the holes on the bottom side of the cover. (Img. 12)
5. Lay the mattress on the underframe. Put the blue part of the cover over mattress and underframe and close the zipper. (Img. 13)

Important hint: If you have a bed with side rails, you will probably need to use side rail extensions. ThevoRelief can also be used without a pre-existing slatted frame. In this case the supporting surfaces along the sides of the bed base must be at least 10 cm (3.9") wide.

6. Pull the supply pipe through the opening of the zipper at the foot section
7. The underframe of ThevoSleepingStar A is safely fixed to the bed frame with the 6 belts. Thread the belts through the bottom holes of the active side element and then around the tubular frame of the bed. Thread the belt through the buckle and pull it tight. To release the belt, open the buckle and pull out the belt. (img 14)

Attention: If you have a height adjustable bed, please fix the belts to the lifting system.

b) Securely fix the control unit to the bed.

c) Plug the line cord of the control unit (230V) into a power socket.

d) Switch the power switch on (light turns red).

Changing the movement pattern

As soon as the light emitting diode (LED) on the front of the device flashes, please press the start button at the control unit and the system will start working. The last chosen program starts running, indicated by the LEDs at the control keys (img. 1, 4).

- a) Stop the currently running movement pattern by pressing the start/stop-switch at the control unit.
- b) Select the now desired new movement pattern (wave, rotation, inclined plane, static) and/or its speed level (fast, medium, slow) by tapping on the corresponding function key. The selection will be confirmed by flashing of the corresponding LED. The function „static“ (bar symbol) is without speed.
- c) Re-activate the system by pressing the start/stop-switch.

Note: In case of an emergency (resuscitation), you can stop the movement pattern by pressing the power switch!

The movement patterns of ThevoSleepingStar A

Due to different movement patterns ThevoSleepingStar A can be adjusted to the child's individual needs. The system runs in an interval of 6 minutes movement pattern and 4 minutes pause. The pause is indicated by flashing of the green "Activ" LED.

The following movement patterns are available:

1. Wave (img. 15)

The actors are approached parallelly right and left. The stimulation pattern is similar to a wave that occasionally flows from the feet to the head and back. The speed can be selected, as desired, in three steps from slow to fast. This stimulation pattern allows body perception of the child from head to foot.

2. Rotation (img. 16)

The actors are approached staggered right and left. This results in reverse wave mountains being globally looked upon as equivalent to a rotation. The direction of rotation is reversed after 3 runs. Here too, the flow speed can be chosen in three steps. The motion of rotation ensures an increased attention with the child. Due to the stimulation of body zones locally lying far apart from each other, a perception of the body is given in its length, but also in its right and left body half.

3. Inclined plane (img. 17)

All actors of one plane are approached simultaneously and thus create an inclined plane. After a previously set time allowance the inclined plane shifts to the other side. With this stimulation pattern the right and left body halves are perceived separately and especially the child's vestibular sense is promoted additionally.

4. Static (img. 18)

This function does not perform any movement pattern. In the interval all actors are evenly lifted.

The throughput speeds can be adjusted, as well:

Fast: 26 sec. Medium: 78 sec. Slow: 120 sec.

Additional valves

As standard ThevoSleepingStar A is equipped with 16 valves, carrying out the selected movement pattern. This provides a good stimulation of the entire body surface. However, certain children may require a more sensible, fine adjusted stimulation to support body perception and mobility. Here, you can use a ThevoSleepingStar A system with 24 valves. Please contact us for further information.

Disinfection methods and cleaning recommendations

Mattress

The covers can easily be taken off due to a sewed on two-sided zipper. The mattresses are autoclavable and therefore suitable for reinstatement.

Incontinence cover

- washable up to 65 °C (149 °F)
- suitable for tumble dryers
- zipper on two sides
- breathable
- to be cleaned by wiping disinfection

Under frame

Easy cleaning: the system can be cleaned with non-corrosive detergent. Please ensure the system to be dried carefully after every cleaning. The system can also be cleaned with disinfectant on alcohol basis.

Disinfection of the under frame

For the disinfection we recommend two methods accepted by the Robert-Koch-Institute:

- the fractionated vacuum method (VDV-method) up to a max. temperature of 110 °C (230 °F).
- the steam flow method up to a temperature of 110 °C (230 °F).

Caution: The control unit must not undergo a steam sterilisation! It can be wiped with a damp cloth, but should in no case be exposed to an open jet of water.

Information: A thermal treatment over 110° C (230 °F) causes premature ageing of the plastic parts and should therefore be avoided. Please make sure, when using chemical disinfectants, that they are suitable for plastic.

Maintenance

In order to ensure a long life cycle and utilisability of the MiS Micro-Stimulation®-System ThevoSleepingStar A, you should follow below stated maintenance instructions:

- Should the red LED „Wrench Symbol“ glow permanently, please call a service technician. Nevertheless the system can be used without any loss of functioning.
- We recommend an annual inspection carried out by qualified personnel.

- The tight fit and impermeability of the actors shall be checked regularly. Leakages, leading to the punctual losses of function are recognised by a loud fizzling. In such cases, tightly press the valve unit in direction of the bellows, whilst supporting the bellow from the exterior with your thumb.

Self test / service LED

Every time the system is switched on (power switch), an automatic self test is carried out, indicating, if applicable, the following errors by the red service LED (wrench-symbol):

- **Continuous flashing**

Not both of the cable harnesses are plugged in.

Note: First, plug in the cable harnesses, then press the power switch!

- **Double flash:**

The number of connected valves on the left and right side does not match.

- **Permanent glow:**

Compressor is due for maintenance. Call the service hotline!

Furthermore:

The service LED also flashes temporarily (3 seconds), if you try to change the movement pattern or speed while the system is operating.

Conditions of warranty

Thomashilfen takes full responsibility for the quality of ThevoSleepingStar A. All materials as well as the processing have been chosen under the highest production standards. All systems have been tested very carefully before leaving the factory. Thomashilfen offers a guarantee of 24 months.

You should therefore carefully keep all documents on ThevoSleepingStar A. The guarantee covers all claims on the basis of production or material failures, which impair functionality. Excepted are wear or tear caused by over or inappropriate use.

Safety hints

Please pay attention to the following points and keep this user manual near to the system, so that you can read it up anytime.

- Only qualified and by company Thomashilfen authorised personnel is allowed to open the control unit. Before opening pull the power plug!
- Keep the control unit away from liquids and open flames.
- In case of damages at the housing or at the supply line, firstly pull the power plug.
- Do not store the system at more than 50 °C (122 °F).
- In certain cases medical electrical instruments can be influenced by mobile phones. If there should be a disturbance by a mobile, the system has to be switched off. Switch it on again after 10 seconds. If the problem still exists, please call the service.
- Don't use ThevoSleepingStar A in humid rooms.
- ThevoSleepingStar A must not be used near to explosive gases.
- In case of damages to the system, do not use it until a repair has been carried out by authorised personnel.

Safety features

Under adverse operating conditions it can happen that the system shuts-off automatically to avoid damage of the device. These safety features apply for short-circuits, disconnection of power and thermal overload.

Short-circuits / interruptions

In case of short-circuits in the actors due to mechanical damage, the short-circuits will be recognised by the system and the wiring concerned will be switched-off. The "Error"-LED at the front of the casing lights up red and the system stops. A re-start is possible by pressing the start button, whereas the defective will be excluded from the information flow, that means this actor is no more functioning. The defective actor has to be exchanged at the first opportunity. The remaining actors and the system remain fully functional. If the defective actor is located close to a "critical caring region", that means directly at the prevention or treatment area, the exchange has to be made as soon as possible.

Thermal overload

In case of high operating temperatures due to unfavourable environmental conditions, as high outside temperature and disturbance of the influx of fresh air, the system switches-off well in advance of damage. A continuing excessive operating temperature inside the system otherwise would lead to early obsolescence. In this case, the operation will be stopped and the red "Error-LED" on the front side of the system will flash. A re-start by pressing the start-button is possible only, if the measured temperature inside the system has fallen below an uncritical threshold value, defined by the manufacturer. The red LED stops flashing and the system is ready for operation again. In case of a repeated occurrence of temperature switching-off, make sure there is a ventilating gap between the back plate of the system and the bed's foot end. Please also make sure this constructively intended gap is not blocked with linen or similar things.

Disposal of the system

ThevoSleepingStar A shall not be disposed with the normal household waste. Please turn to registered disposal companies or return the system to your specialist dealer.

Technical data

| Item Code | 61301 | 61311 | 61361 | 65302 | 65322 |
|----------------------------|-----------------------------|------------------------------|-----------------------------|-----------------------------|------------------------------|
| Size | 60 x 120 cm 23.6 x 47.2" | 70 x 140 cm 27.6 x 55.11" | 90 x 170 cm 35.4 x 66.9" | 90 x 200 cm 35.4 x 78.7" | 100 x 200 cm 39.4 x 78.7" |
| Active side elements (ASE) | 14 | 16 | 20 | 24 | 24 |
| Flexible fibreglass-tracks | 8 | 9 | 11 | 13 | 13 |
| Wing suspensions | 16 | 23 | 39 | 46 | 52 |
| Height | 19 cm/7.5" | 19 cm/7.5" | 19 cm/7.5" | 19 cm/7.5" | 19 cm/7.5" |
| Weight | 12.9 kg/28.44 lbs | 16.6 kg/36.6 lbs | 20.9 kg/46.1 lbs | 21.8 kg/48.1 lbs | 23.6 kg /52 lbs |

Other measurements on request.

Note: The mattress specially developed for the needs of children is suitable for a body weight up to 60 kg / 132.3 lbs. For a body weight of more than 60 kg / 132.3 lbs we recommend using a positioning system for adults. We are happy to offer our assistance.

Control unit

Operational voltage: 230 V ~ 50 Hz
 Max. power consumption: 50 W
 Protection class I: IPX 0
 Measurements of control unit: 38 x 21 x 21 cm (14.96 x 8.24 x 5.24")
 Medical device: Class 1 acc. to guideline 93/42/EEC

Handwriting practice lines consisting of 30 horizontal lines.

Member of the International
Support Association for the
Rehabilitation of Children and Youths e.V.

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