
# ThevoSchlummerstern Senso

ⓁⒹ	<b>Bedienerhandbuch – ThevoSchlummerstern Senso</b> .....	<b>3</b>
ⓁⒹ	<b>User manual – ThevoSleepingstar Senso</b> .....	<b>6</b>

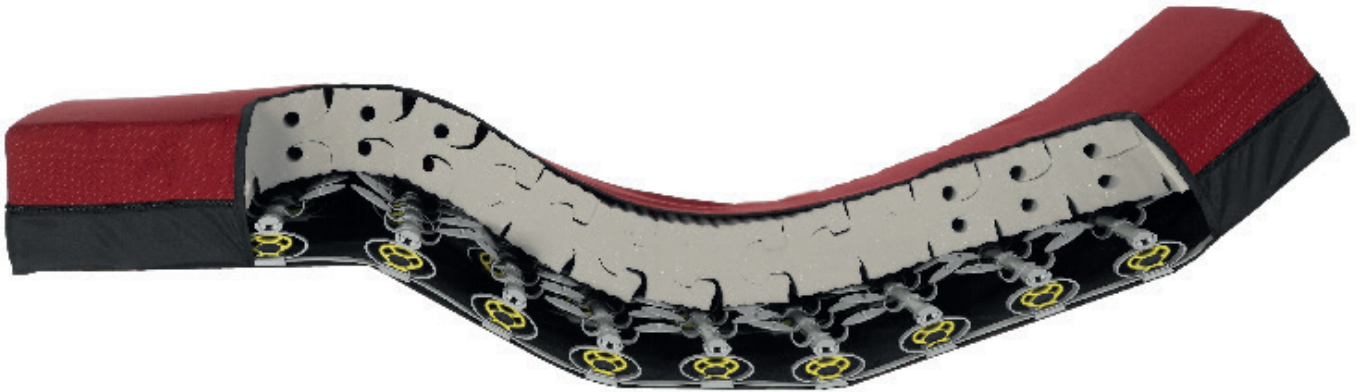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## Dear User,

You have just acquired a product - the **Thevo**Sleepingstar Senso mobility bed - that meets the highest quality standards. **Thevo**Sleepingstar Senso features the latest materials for an extended service life that's guaranteed.

Please read through this manual carefully before using **Thevo**Sleepingstar Senso. It contains lots of useful information and tips.

If you have any further questions or suggestions concerning **Thevo**Sleepingstar Senso, feel free to give us a call.

Further information is also available at on [www.thevo.info](http://www.thevo.info)

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## Contents

	Page
Technical data .....	7
Scope of delivery.....	7
Usage.....	7
Disinfection methods and cleaning recommendations .....	7
Application risks and contraindications.....	8
Warranty conditions.....	8

## Technical data

ThevoSleepingstar Senso	60 x 120 cm (23.6 x 47.2 ")	70 x 140 cm (27.6 x 55.1 ")	90 x 200 cm (35.4 x 78.7 ")	100 x 200 cm (39.4 x 78.7 ")	Height
ThevoSleepingstar Senso	16 - 22 kg / 35 - 49 lbs*				19,0 cm (7.5 ")

\*Relates to the mattress sizes mentioned above. Weight can vary minimally, because of production tolerance.

Other dimensions available on request.

## Scope of delivery

- ThevoSleepingstar Senso with jersey or incontinence cover
- User manual

## Usage

The product is compatible for use with every type of bed. Lay **ThevoSleepingstar Senso** with the red side facing upwards on the existing bed slats or bed base.

**Important hint:** ThevoSleepingstar Senso can also be used without a pre-existing slatted frame. In this case the supporting surfaces along the sides of the bed base must be at least 13 cm / 5.1" wide.

**Attention:** Beds with side-rails may require additional side-rail height extenders.

## Disinfection methods and cleaning information

### Mattress

The washable covers are easy to remove thanks to an integrated zip.

### Jersey cover (fabric cover)

- washable at 65 °C (149 °F) • suitable for tumble dryers

### Incontinence cover (rubberised cover)

- washable at 65 °C (149 °F) • suitable for tumble dryers • to be cleaned by wiping disinfection

### Disinfection

We recommend two disinfection methods accepted by the Robert Koch Institute:

- the vacuum/steam/vacuum method up to a maximum temperature of 110 °C (230 °F)
- the steam flow method up to a temperature of 110 °C (230 °F)

**Important hint:** A thermal treatment over 110 °C (230 °F) causes a premature ageing and deformation of the plastic parts and should therefore be avoided. Please make sure when using chemical disinfectants that they are suitable for plastic.

## Application risks and contraindications

**Thevo**Sleepingstar Senso was designed to support perception and improve the quality of sleep of children with perception and behavior disorders.

The use of **Thevo**Sleepingstar Senso does not require any special instructions. For an optimal therapeutic outcome, the system needs to be adapted to the individual needs of the child. After practical use and conducting several tests, we are unaware of any problems associated with the use of this product. When used properly, there are no known risks.

**Note: ThevoSleepingstar Senso is not suitable for the prevention and treatment of pressure ulcers. Here we recommend using a special mobility bed. We will be glad to advise you.**

## Warranty conditions

The company Thomashilfen takes full responsibility for the quality of the product supplied. The highest production standards are used in the selection and processing of materials. All **Thevo**Sleepingstar Senso mattresses are subjected to a thorough final inspection. Thomashilfen offers you a warranty of 24 months. Please retain all relevant **Thevo**Sleepingstar Senso receipts. The warranty covers all claims arising from manufacturing or material defects that affect product function. Damage arising from inappropriate use and natural wear and tear is excluded.

**Important hint: ThevoSleepingstar Senso is autoclavable and may be reused. ThevoSleepingstar Senso should be cleaned and disinfected using one of the cleaning methods stated above with every change of user. Servicing by the manufacturer is not necessary.**

Member of the International  
Support Association for the  
Rehabilitation of Children and Youths e.V.

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Walkmühlenstraße 1 · D - 27432 Bremervörde  
Telefon: +49 (0) 47 61 / 8 860 · Fax: +49 (0) 47 61 / 8 86-19  
E-mail: info@thomashilfen.de · www.thevo.info