ThevoSleepingStar

ⓁⒹ	Bedienerhandbuch – ThevoSleepingStar	3
ⓁⒸ	User manual – ThevoSleepingStar	6
ⓁⓇ	Manuel d’utilisation – ThevoSleepingStar	9
ⓁⓈ	Manual de usuario – ThevoSleepingStar	12
ⓁⓉ	Manuale d’istruzioni – ThevoSleepingStar	15
ⓁⓃ	Gebruiksaanwijzing – ThevoSleepingStar	19
ⓁⓃ	Bruksanvisning – ThevoSleepingStar	21
ⓁⓈ	Bruksanvisning – ThevoSleepingStar	24
ⓁⓀ	Brugsanvisning – ThevoSleepingStar	27
ⓁⓂ	Käyttäjän käsikirja – ThevoSleepingStar	30

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Dear User,

You have just acquired a therapy system for ill and disabled children - the ThevoSleepingStar mobility bed - that meets the highest quality standards. ThevoSleepingStar features the latest materials for an extended service life that's guaranteed. Please read through this manual carefully before using ThevoSleepingStar. It contains lots of useful information and tips. If you have any further questions or suggestions concerning ThevoSleepingStar, do not hesitate to call us.

Further information is also available from www.thevo.info

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Exomotion, LLC
309 South Cloverdale Street Unit B 12
Seattle WA 98108 USA
Phone: 206 763 0754 Intel: 858.712.0840
E-Mail: info@exomotion.com
www.thevo.info

Contents

	Page
Sleep disorders of children with special needs	7
Fields of application and clinical pictures	7
Sizes and components.....	7
Technical data	7
Scope of delivery.....	7
Usage.....	7
Disinfection methods and cleaning information	7
Application risks and contraindications.....	8
Warranty conditions.....	8

Sleep disorders of disabled children

ThevoSleepingStar is a positioning and therapeutic system suitable for children, which has been developed to ease the nightly care of ill and disabled children.

It can be used for the perception support of disabled children, for the support of a good pain therapy, for the development of a physiological sleep pattern, for the inhibition of muscle tensions and for the prevention and treatment of pressure ulcers.

Fields of application:

- Perception-, movement-, and development support
- Treatment of sleep disorders of disabled children
- Prevention of contractures
- Inhibition of spasticity
- Support of pain therapy
- Preservation of body image
- Supports / promotes a physiological sleep architecture
- Prevention and treatment of pressure ulcers

Clinical pictures:

Pain, coma vigil, coma, craniocerebral injury, spina bifida, cerebral palsy and similar clinical pictures, oncological diseases, genetic skin diseases.

Technical data

Outside dimensions ThevoSleepingStar	60 x 120 cm (23.6 x 47.2 ")	70 x 140 cm (27.6 x 55.1 ")	90 x 200 cm (35.4 x 78.7 ")	100 x 200 cm (39.4 x 78.7 ")
Weight ThevoSleepingStar	about 16 - 22 kg / 35 - 49 lbs*			
Height ThevoSleepingStar	19.0 cm (7.5 ")			

Other dimensions available on request.

Note: The mattress was developed especially for the needs of children and is suitable for a body weight up to 60 kg / 132.3 lbs. For a body weight of more than 60 kg / 132.3 lbs we recommend using a mobility bed for adults. We are happy to offer our assistance.

Scope of delivery

- ThevoSleepingStar with incontinence cover
- User manual

Usage

The product is compatible for use with every type of bed. Lay ThevoSleepingStar with the red side facing upwards on the existing bed slats or bed base.

Important hint: ThevoSleepingStar can also be used without a pre-existing slatted frame. In this case the supporting surfaces along the sides of the bed base must be at least 13 cm (5.1 ") wide.

Attention: Beds with side-rails may require additional side-rail height extenders.

Disinfection methods and cleaning information

Mattress

The washable cover is easy to remove thanks to an integrated zip.

Incontinence cover (rubberised cover)

- washable at 65 °C (149 °F)
- suitable for tumble dryers
- to be cleaned by wiping disinfection

Disinfection

We recommend two disinfection methods accepted by the Robert Koch Institute:

- the vacuum/steam/vacuum method up to a maximum temperature of 110 °C (230 °F)
- the steam flow method up to a temperature of 110 °C (230 °F)

Note: Please contact us, if you intend using one of the disinfection methods above. We will be happy providing you with detailed instructions.

Important hint: A thermal treatment over 110 °C (230 °F) causes a premature ageing and deformation of the plastic parts and should therefore be avoided. Please make sure when using chemical disinfectants that they are suitable for plastic.

Application risks and contraindications

ThevoSleepingStar was designed to support perception and improve the quality of sleep of children with perception and behavior disorders.

The use of ThevoSleepingStar does not require any special instructions. For an optimal therapeutic outcome, the system needs to be adapted to the individual needs of the child. After practical use and conducting several tests, we are unaware of any problems associated with the use of this product. When used properly, there are no known risks.

Important Note: Make sure to select the mobility bed appropriate for the body weight of the child. Thevo SleepingStar for a body weight of up to 60 kg / 132.3 lbs. At the threshold of 60 kg/132.3 lbs (+/- 2kg / 4.4 lbs) a ThevoautoActiv must be used. We are happy to advise you.

Warranty conditions

The company Thomashilfen takes full responsibility for the quality of the product supplied. The highest production standards are used in the selection and processing of materials. All ThevoSleepingStar mobility beds are subjected to a thorough final inspection. Thomashilfen offers you a warranty of 24 months. Please retain all relevant ThevoSleepingStar receipts. The warranty covers all claims arising from manufacturing or material defects that affect product function. Damage arising from inappropriate use and natural wear and tear is excluded.

Important hint: ThevoSleepingStar is autoclavable and may be re-used. ThevoSleepingStar should be cleaned and disinfected using one of the cleaning methods stated on page 7 and 8 with every change of user. Servicing by the manufacturer is not necessary.

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