ThevoMulti

Ⓧ	DE	Bedienerhandbuch – ThevoMulti.....	4
Ⓧ	GB	User manual – ThevoMulti	8
Ⓧ	FR	Manuel d’utilisation – ThevoMulti.....	12
Ⓧ	ES	Manual de usuario – ThevoMulti.....	16
Ⓧ	IT	Manuale d’istruzioni – ThevoMulti.....	20
Ⓧ	NL	Gebruiksaanwijzing – ThevoMulti	24
Ⓧ	NO	Bruksanvisning – ThevoMulti.....	28
Ⓧ	SE	Bruksanvisning – ThevoMulti.....	32
Ⓧ	DK	Brugsanvisning – ThevoMulti.....	36
Ⓧ	FI	Käyttäjän käsikirja – ThevoMulti	40

(1)

(2)

(3)

(4)

(5)

(6)

(7)

(8)

(9)

(10)

(11)

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Dear User,

You have acquired in a ThevoMulti product that meets the highest quality standards. State-of-the-art materials were used for the ThevoMulti which guarantee a long service life.

Before you start using your ThevoMulti product, read through this user manual carefully because it will provide you with helpful information and tips.

If you have questions or suggestions regarding the ThevoMulti, feel free to give us a call.

You will find more information at **www.thevo.info**

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Contents	Page
Safety Instructions and Warning	9
Delivery Scope	9
Technical Data	9
Chair Set Up	10
Basic Settings.....	10
Basic Equipment	10
Accessories.....	11
Disinfection Methods and Cleaning Recommendations.....	11
Warranty Conditions.....	11

Safety Instructions and Warning

Please do not overload your ThevoMulti and take note of the maximum weight (see technical data).

Warning: Never put fingers in the recliner mechanism. Make sure there are no people or objects near the ThevoMulti's recliner mechanism. This is dangerous.

The ThevoMulti should be placed and used on level surfaces only. Do not use the ThevoMulti in areas that are exposed to water and very hot temperatures.

Avoid placing heavy objects on the ThevoMulti. Do not sit or stand on the armrests or backrest of the ThevoMulti.

Do not sit or stand on the footrest (accessory) because the surface is slanted and there is a risk of sliding off.

Under no circumstances should cords be jammed, put near sharp edges or squished by the moveable seat parts.

The recliner motor should not be operated for more than two minutes at a time. Thereafter the motor should not be used for roughly 18 minutes. Keep this manual in a safe place! Only original spare parts from the manufacturer should be used. Don't unscrew or remove any screws unless indicated.

Delivery Scope

- ThevoMulti
- User Manual
- Hand control
- Transformer and power cord
- Cover for the base of the chair
- Neck support pillow
- Accessories, as required

Technical Data

ThevoMulti	
Overall height	119 cm / 49, 8"
Width (overall)	73,5 cm / 28,9"
Seat depth	45 cm / 17,7"
Seat width	55 cm / 21,6"
Seat height	48 - 54 cm / 18,9" - 21,3"
Angle of seating position	103°
Angle of resting position	131°
Maximum load	140 kg / 308,65 lbs
Footrest (Accessory)	
Width	41 cm / 16,1"
Width	44 cm / 17,3"
Depth	44 cm / 17,3"
Transformer	
Input	230V 50Hz alternating current
Protection class	II
IP code	IP20
Motor	24V direct current

Chair Set Up

After you have unpacked the ThevoMulti it just needs to be plugged in. Do this as follows:

1. Check that the power plug and the ThevoMulti cord are each plugged into the transformer.
2. Then check whether the cord from the hand control is connected with the motor of the ThevoMulti.
3. Connect the power plug to the power supply.

Now your ThevoMulti is ready to use. If the ThevoMulti is lifted by its side parts after set up, the backrest might fold back. This is because of its design and is not a product defect. To prevent this from happening, support the backrest of the ThevoMulti whenever you lift it.

Basic Settings

Use the hand control to change the position of the ThevoMulti. The hand control has two buttons. After attaching the back rest, the ThevoMulti is in a high position.

To return to the base position, use the right button (red symbol) on the hand control. Press down on the button until the ThevoMulti reaches the starting position. (Picture 2)

To move the ThevoMulti from the base position to the standing up position, press down on the left button (blue symbol) on the hand control. The ThevoMulti will then go into a higher sitting position. (Picture 3) At this point it is still possible to use the revolving function.

Then the ThevoMulti goes into the position for standing up. (Picture 4) At this point the revolving function locks.

To reach the resting position press down on the right button (red symbol) on the hand control. After reaching the base position, the backrest reclines until the seat is in the resting position. (Picture 5)

Basic Equipment

Attaching the cover for the base of the chair

You can attach the cover for the base of the chair as follows:

1. Carefully tip the ThevoMulti on its side and put on a clean underlay.
2. Undo the 4 zippers of the cover.
3. Pull the cover over the base of the chair making sure that all the feet are visible and then do up all the zippers.
4. Adjust the cover so it is even.
5. Carefully stand the ThevoMulti up again. – Done

Attaching the neck support pillow

On the neck support pillow there is a strip of material weighted at the end. To attach the neck support pillow to the ThevoMulti, proceed as follows:

1. Position the neck support pillow in the head area of the backrest of the ThevoMulti. When doing so, the strip of material on the underside of the neck support pillow has to be facing up.
2. Put the strip of material over the backrest of the ThevoMulti. It should hang down from the backrest of the chair.
3. The neck support pillow is simply adjusted in height for ideal positioning . (Picture 11)

Accessories

We offer the following accessories for the ThevoMulti:

- Footrest
- Incontinence covers

Attaching the incontinence covers

The set consists of five individual protective covers for the backrest, the seating surface and both side parts of the ThevoMulti and for the top of the footrest. We recommend using the protective covers particularly if it is used frequently such as in inpatient care settings.

See Picture	Tips for Attaching
6	Pull the cover over the backrest . Make sure the side seam of the cover is placed directly over the side seam of the backrest. Close the Velcro of the cover at the bottom of the backrest on the left and right of the metal housing.
7	Pull the cover over the seating area. Make sure the longer fold of the cover is on the front of the seat.
8 + 9	Pull the covers over the left (Picture 8) and right sides (Picture 9) of the ThevoMulti. Make sure the long slits of the cover are each on the inside of the ThevoMulti at the back. Press the harder, sewn fabric edge located at the upper area of the cover from the outside between the armrest and side part. The small slits in the cover are for the bracket of the hand control holder.
10	Pull the cover over the surface of the footrest .

You can order accessories for the ThevoMulti from Thomashilfen. We are pleased to assist you.

Disinfection Methods and Cleaning Recommendations

You have purchased a product made from high quality materials that requires proper care. Long and intensive exposure to the sun can lead to fading.

The covers cannot be washed separately. Never try to take off the covers! Clean the ThevoMulti with disinfectant wipes. To clean the hand control, unplug the ThevoMulti first. The hand control can be cleaned with a damp cloth and standard plastic cleaner.

If the chair is used frequently, such as in inpatient care settings, we recommend using our protective covers.

Warranty Conditions

Thomashilfen guarantees the quality of the delivered product. Highest production standards were applied when selecting the material and during processing. Your TheoMulti product was subject to a rigorous final inspection.

Thomashilfen offers you a two-year warranty. Please keep all of the receipts for the ThevoMulti in a safe place. The warranty covers all claims which arise from workmanship and material flaws which impact the chair's function. Exceptions include damages as a result of incorrect handling and natural wear.

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